maxEVENT

Bedienungsanleitung
User Manual
Mode d'emploi
Istruzioni d'uso
Manual de instrucciones
Gebruiksaanwijzing
Betjeningsvejledning
Brugsanvisning
Bruksanvisning
Käyttöopas
Instrukcja obsługi
Provozní návod
サービスマニュアル

CE

ESCHENBACH

Deutsch	4
English.....	7
Français.....	10
Italiano.....	13
Español.....	16
Nederlands.....	20
Dansk	23
Svenska.....	26
Norsk.....	29
Suomi.....	32
Polski.....	35
Česky	39
日本語.....	42

Deutsch

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English

Your new acquisition is a quality product from the house of Eschenbach. This top-class "Made in Germany" product was manufactured using the most up-to-date production methods available. Congratulations on your choice.

With MaxEvent you can see things twice as large at events such as concerts, and in the cinema or theatre. The focus of its design was on user-friendliness and a modern style.

Safety instructions

- **Danger of blinding or injury!**
MaxEvent are not sunglasses! The metallic coating of the lenses do not offer protection from the sun's rays! Never look at the sun or other very bright sources of light through optical equipment!
- **Risk of fire!**
Lenses used in optical equipment may cause considerable damage by their "burning glass effect" if used or stored improperly. Make sure that optical lenses are never left uncovered in the sunlight.
- **Make sure that other people and especially children are aware of this.**
- **MaxEvent must not be used for driving in traffic!**
- **Protect your MaxEvent from impact and excessive heat! Never place the MaxEvent on heaters or in sunlight.**

Earpiece adjustment

The plastic earpieces can be bent in the desired position without warming. An internal metal wire ensures that the earpieces remain in this position.

Adjustment

The MAXEvent is designed for a distance of about 10 m and offers a dioptre adjustment (option for correcting defective vision) of -2.75 dpt to +3.75 dpt. It can be adjusted for each eye individually. You can also use the dioptre adjustment to adapt the glasses to various distances.

Position yourself at the desired distance and put on your MAXEvent.

Keep your left eye closed and turn the knurled ring [1] until your right eye can see in sharp focus. Do the same for the other eye.

Notes

Under the following circumstances, it might not be possible to see the image perfectly:

- If the MAXEvent is bent open too far, for example due to an unusually large head, doubled images might occur, because the optical axes of the two lens systems are not in parallel anymore.
- A very small or very large pupil spacing (eye spacing) can reduce the performance quality, because the image is not viewed through the optical zones of the MAXEvent anymore.
- If you normally have glasses with dioptres of less than -2.75 dioptres or greater than +3.75 dioptres, the dioptre adjustment feature of the MAXEvent cannot entirely compensate the defective vision.

- If you usually wear glasses with cylindrical lenses (e. g. for astigmatism), it might not be possible to achieve perfect focus. This depends on the strength of the cylinder.

Care instructions

MAXEvent is a high-quality, technologically advanced, complex optical aid and needs to be handled and cared for accordingly. For protection of the mirror coating and the lenses against dust and external influences, we recommend that you store it in the case included.

- **Do not use alcohol or organic solvents to clean the MAXEvent.**
- **Do not clean the MAXEvent in an ultrasonic bath!**

Clean the MAXEvent with a soft cotton or linen cloth (such as cleaning cloths for eyeglasses). Moisten the cleaning cloth to remove more persistent stains (e.g. finger marks). Do not rub the sensitive mirror coating with a dry cloth.

Technical data

Article number	162431
Dioptr adjustment for each eye individually	-2.75 dpt to +3.75 dpt
Magnification	2,1 x
Field of vision	320 m / 1000 m
Pupil spacing	60 mm ... 68 mm
Weight	49 g

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Garantee

We are required by law to guarantee the function of the product described in these descriptions in respect of defects caused by manufacturing or material faults. Any damages caused by incorrect handling, including damage caused by being dropped or knocked, are not covered under the guarantee. Guarantee benefits can only be claimed through proof of purchase in the form of a sales receipt!

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Eschenbach Optik GmbH